



# Supervising a Sound Scouts Test

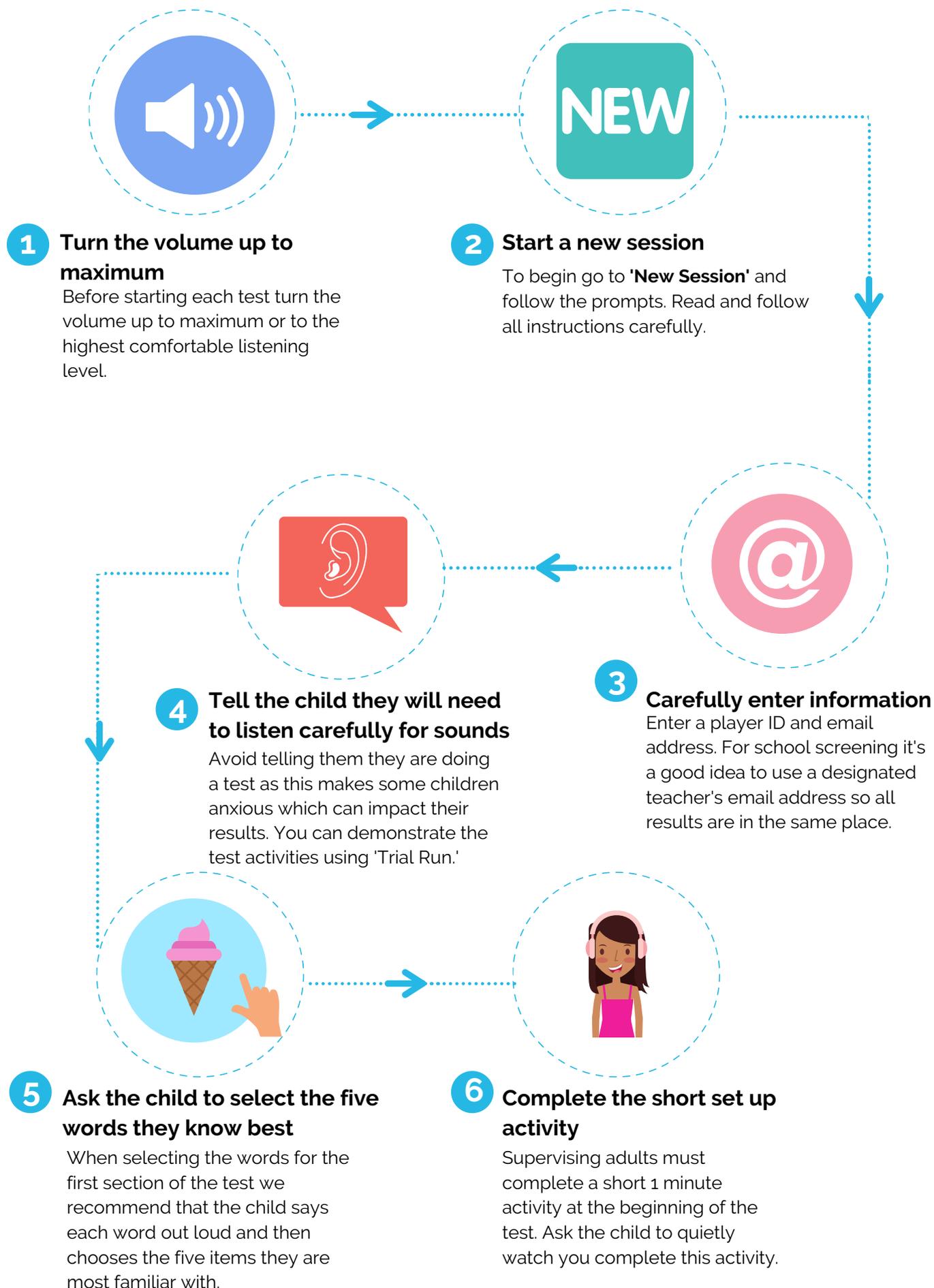



### 7 Place the headphones on the child

After the set up activity is complete, place the headphones on the child being tested. Make sure the headphones are sitting correctly on or over the ears.

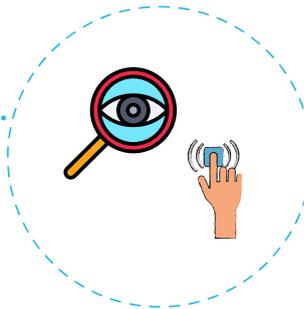

### 8 Watch for excessive tapping

During the test the adult should watch carefully to ensure the child doesn't tap repeatedly when no sounds are being played. If this happens explain that they should only tap the screen when they hear a sound.




### 10 If a big disruption occurs, restart the test

If anything happens during the test that may impact results (eg. persistent loud noise for more than a minute) stop the test and start again.



### 9 If the child is not interacting you can intervene

In this case, ask the child what they hear and tap or slide the matching object for them. Demonstrating once or twice is usually enough. If further assistance is required you may wish to provide further explanation to the child and then **restart** the test.

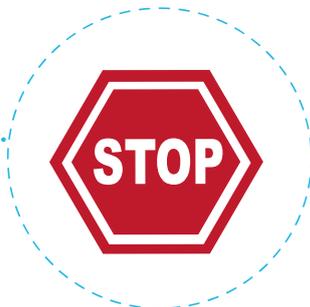

### 11 To stop the test early you will need to close the app


To STOP the test early (if the child needs to start again) you will need to close the App using the method appropriate to the device (i.e. Apple vs Android). Typically you swipe up to close the App.

# After the Test



## 1 Results are delivered immediately

You can view the results on the device and they will also be automatically emailed to the address entered at the beginning.

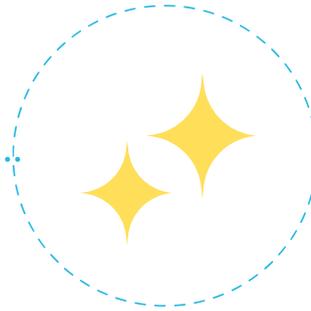

## 2 Clean the device and headphones

Wipe down screens and headsets and replace headphone protectors if applicable.




## 4 Stored results can be accessed through 'Past Sessions'

To retrieve a session go to 'Past Sessions,' select the session and if a connection is available the results will be processed.



## 3 If you don't receive results...

If the device is not connected to a wifi network the test data will be stored and can be processed when a connection becomes available. You will not receive results until you connect to wifi.

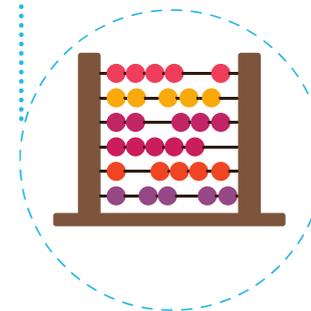

## 5 Results won't be registered if the test session was incomplete

A test session is not used or registered until the test is complete.



## 6 Read the report carefully and follow the next steps

We suggest following the recommended actions in the report. If a child fails the first test we recommend a re-test within the week to confirm results and next steps.